







# TECHNOPLASTIN® HIS

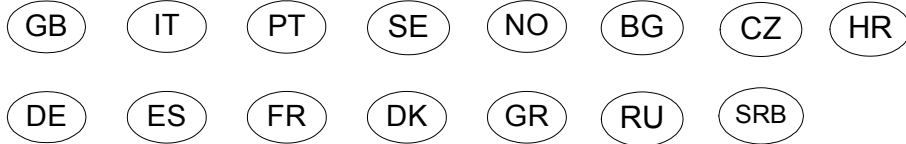

For Research Use Only

REF	5003009	TECHNOPLASTIN® HIS	12 x 2 mL
REF	5003010	TECHNOPLASTIN® HIS f. Ceveron	12 x 4 mL
REF	5003030	TECHNOPLASTIN® HIS	2 x 10 mL
REF	5003026	TECHNOPLASTIN® HIS	6 x 10 mL
REF	5003028	TECHNOPLASTIN® HIS f. Ceveron	6 x 20 mL
REF	5003021	TECHNOPLASTIN® HIS	20 x 10 mL
REF	5003022	TECHNOPLASTIN® HIS f. Ceveron	20 x 20 mL

## symbols key / Symbolschlüssel / interpretazione dei simboli / explicación de símbolos / explicação dos símbolos / clé des symboles / Symbolnyckel / symbolforklaring / Tegnforklaring / Κλειδί συμβόλων / Използвани символи / символы / Klíčova slova / Značenje simbola

	manufacturer / Hersteller / fabbricante / fabricante / fabricante / fabricant / Tillverkaren / Fabrikanten / Produsent / Κατασκευαστής / Производитель / дистрилированная вода / destilovaná voda / Destilšana Voda		expiry date / Verfallsdatum / data di scadenza / fecha de caducidad / data de validade / date d'expiration / utgångsdatum / udløbsdato / Utløpsdato / Ημερομηνία λήξης / срок на годност / datum expirace / срок годности / datum expirace / Rok trajanja
	storage temperature / Lagertemperatur / temperatura di conservazione / temperatura de conservación / temperatura de conservação / température de stockage / lagringstemperatur / oppbevaringstemperatur / Oppbevaringstemperatur / θερμοκρασία αποθήκευσης / хранение на / teplota skladování / температура хранения / teplota skladování / Temperatura lagerovanja		consult instructions for use / Gebrauchsanweisung beachten / consultare le istruzioni per l'uso / consulte las instrucciones de uso / consultar o manual de instruções / instruction d'utilisation / se användarinstruktioner / følg brugsvejledning / Følg bruksanvisningen / συμβουλευθείτε τις οδηγίες για τη χρήση / прочетете инструкцията за работа / potfeba fidiť se instrukciami / перед использованием читайте инструкцию / sledujte návod k použití / Pročitaj upustvo pre upotrebe
			determinations / Bestimmungen / determinazioni / determinaciones / determinações / détermination / bestämmingar / bestemmelse / Bestemmelse / προσδιορισμοί / брой тестове / stanovení / определный / ročet stanovení / Definicija
<b>AQUA</b>	distilled water / destilliertes Wasser / acqua distillata / agua destilada / água destilada / eau distillée / destillerat vatten / destilleret vand / Destillert vann / απεσταγμένο νερό / дестилирана вода / destilovaná voda / дистрилированная вода / destilovaná voda / Destilšana Voda	<b>LOT</b>	lot / Charge / lotto / lote / lote / lot / sats / serie / Parti / παρτία / партида номер / šarže / lot / šarže / Serija
<b>BUF</b>	Reaction buffer / Reaktionspuffer / tampone di reazione / tampón de reacció / Tampão de reação / tampon de réaction / Reaktionsbuffert / Reaktionsbuffer / Reaksjonsbuffer / διάλυμα αντίδρασης / Реакционен буфер / Рабочий буферный раствор / Reakční pufr / Reakcioni pufer	<b>MTP</b>	microtiter plate / Mikrotiterplatte / placa microtiter / microplaca / microplaca / microplaques sensibilisées / Mikrotiterplatta / Mikrotiterplade / mikrotiterplate / πλάκα μικροπιλοδότησης / Микротительна пласка / Микропланшет / Mikrotitracijska destička / Mikrotitracione ploče
<b>CAL</b>	Calibrator / Kalibrator / Calibratore / calibrador / calibrador / calibreur / Kalibrator / Kalibrator / Kalibrator / Βαθμονομητής / Калибратор / калибратор / kalibrátor / Kalibrator	<b>REF</b>	catalogue number / Katalognummer / numero di catalogo / número de catálogo / número de referência / réf. de catalogue / katalognummer / Katalognummer / αριθμός καταλόγων / каталожен номер / katalogové číslo / каталожный номер / katalogové číslo / Kataloški broj
<b>CONJ</b>	Conjugate / Konjugat / Coniugato / conjugado / conjugado / conjugaté / Konjugerad / Konjugat / Konjugat / συνδέτικό / Конюгат / Конъюгат / Konjugát / Konjugat	<b>RTU</b>	ready to use / gebrauchsfertig / pronto all'uso / listo para usar / pronto a usar / prêt à l'emploi / færdig att användas / færdig til brug / klar til bruk / έτοιμο προς χρήση / Готов за употреба / готов к использованию / k přímému použití / Razrediti ili rastvoriti
<b>CONT</b>	Control / Kontrolle / controllo / control / control / contrôle / Kontroll / Kontroll / Kontroll / διάλυμα ελέγχου / Контрол / Контрольный образец / Kontrola / Kontrola	<b>STOP</b>	stop solution / Stoppløsning / Soluzione di arresto / solución de parada / solução de paragem / solution d'arrêt / Stoppløsning / Stop-opløsning / Stoppløsning / διάλυμα παύσης / Стоп раствор / Стоп-раствор / Zastavovací roztok / Stop solucija
<b>DIL</b>	dilute or dissolve in / verdünnen oder lösen in / diluire o dissolvere in / diluir o dissolver / diluir ou dissolver em / diluer ou dissoudre dans / späd eller upplös i / fortyndes eller opløses i / Fortyndes eller oppløses i / αραιωση ή διάλυση σε / растворяете или разрежете с / zředit anebo rozpustit v / разбавить или растворить в / nafedite nebo rozpustite v / razrediti ili rastvoriti u	<b>SUB</b>	substrate / Substrat / substrato / substrato / substrato / substrat / Substrat / Substrat / Substrat / υπόστρωμα / Субстрат / Субстрат / Substrát / Substrat
<b>INC</b>	incubation buffer / Inkubationspuffer / tampone di incubazione / tampón de incubación / tampão de incubação / tampon d'incubation / Inkubationsbuffert / Inkubationsbuffer / Vaskebufferkonsentrat / διάλυμα επώασης / Инкубационен буфер / Буфер для инкубации / Inkubační pufr / Inkubacioni pufer	<b>WASH</b>	washing solution concentrate / Waschlösungskonzentrat / concentrado de solución de lavado / solución de lavado concentrada / tampão de lavagem concentrado / Tampon de lavage concentré / Vattenlösingskoncentrat / Vaskeopløsningskoncentrat / vaskeløsningskoncentrat / συμπυκνωμένο διάλυμα πλύσης / Концентриран миеш раствор / Концентрат промывочного раствора / Koncentrát promývachio roztoku / Koncentrat solucije za ispiranje
<b>RUO</b>	for research use only		















