



PL Chip for T-TAS[®] 01

For the measurement of primary hemostatic function in whole blood where clinically significant bleeding is possible.

Primary hemostasis can have a major impact on:



Blood Product Management



Trauma



Interventional Procedures



Surgery

Measuring primary hemostasis can help evaluate if:

- The patient might benefit from platelet transfusion
- Pre-surgical primary hemostasis has been restored
- Active bleeding is associated with impaired platelet activity
- Platelet activity is significantly impaired in association with interventional procedures
- Antiplatelet therapy is showing the desired effect



The technology:


- Uses physiological arterial shear stress to assess platelet thrombus formation in whole blood
- Contains flow chamber with 26 collagen-coated microcapillaries
- Generates results within 10 minutes of start and within 40 minutes of sample collection
- Accommodates 2 samples per chip



Magnified View

The T-TAS 01[®] PL assay selectively measures primary hemostatic function:

- Not influenced by secondary hemostatic function
- Significant dose-response correlation with intensity of antiplatelet therapy, vWF antigen, and vWF activity levels

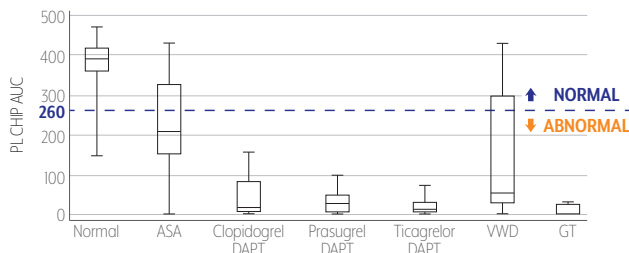

It's not just support. It's solutions.


Visit diapharma.com for our full line of hemostasis clinical and research products.

AR Chip* for T-TAS® 01

For the Measurement of Overall Hemostatic Function in Whole Blood

An *in vitro* measurement of white thrombus formation (i.e., fibrin-rich platelet thrombus)

- Uses citrated whole blood at shear rates representative of large arteries
- Contains a microcapillary channel coated with collagen and tissue thromboplastin
- Measures the combined effect of anticoagulation and antiplatelet agents on overall hemostatic function

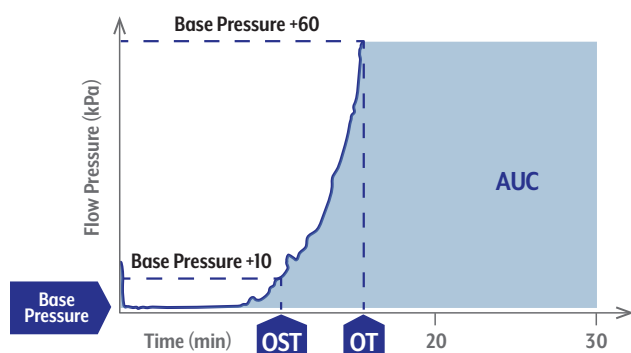

Clinical and therapeutic research areas:

- Direct oral anticoagulant (DOAC) monitoring
- Assessment of bleeding risk
- Coagulation factor therapy
- Thrombolysis/fibrinolysis

Ordering Information

Item	Catalog Number
T-TAS® 01 Total Thrombus Formation Analysis System Instrument	18001
PL Chip for T-TAS® 01 (20 Chips)	18002
PL Chip Reservoir Set for T-TAS® 01 (100 Sets)	18003
BAPA Tube for T-TAS® 01 (50 Tubes)	18004
AR Chip for T-TAS® 01 (20 Chips)	19001
HD Chip for T-TAS® 01 (20 Chips)	19002
AR and HD Chip Reservoir Set for T-TAS® 01 (100 Sets)	19003
CaCTI Reagent for T-TAS® 01 (20 Assays)	19004

Questions about any product in our lineup?

We're just a click or phone call away.

800.526.5224 | info@diapharma.com | diapharma.com

*Research use only in the U.S. and Canada. Not for use in diagnostic procedures.

HD Chip* for T-TAS® 01

For the measurement of overall hemostatic function in whole blood with low platelet count.

An *in vitro* measurement of white thrombus formation (i.e., fibrin-rich platelet thrombus)

- Uses citrated whole blood at arterial shear rate
- Contains a microcapillary channel coated with collagen and tissue thromboplastin
- Measures overall hemostatic function in whole blood samples with low platelet count (as low as 10,000/ μ L)

Clinical and therapeutic research areas:

- Thrombocytopenia
- Assessment of bleeding risk
- Platelet transfusion
- Hemostasis management

Specifications

Sample Type	PL Chip: BAPA Anticoagulated Whole Blood AR and HD Chip: Sodium Citrated Whole Blood
Sample Volume	320 μ L
Test Duration	PL Chip: 10 Minutes AR and HD Chip: 30 Minutes
Sample Stability	Up to 6 Hours
Reagent Storage	Assay Chips: 4-8°C CaCTI: 2-8°C BAPA Tube: 15-30°C
Open Pouch Stability	Up to 8 Hours
Instrument Dimensions	14.2"x12.6"x9.7" (36x32x24.7)
Instrument Weight	13.2 lbs (6.0kg)
Operating Conditions	18-30°C 20-80% Relative Humidity



ML-08-00032Rev01