

TECHNOVIEW Arixtra CAL - English

INTENDED USE

Technoview Arixtra CAL is a set of 5 calibration plasmas to be used for calibration of Arixtra measurements, optimized using Technochrom anti-Xa assay.

SUMMARY

Fondaparinux sodium (Arixtra) is an anticoagulant, used as an alternative therapy to Heparin or low molecular weight Heparin (LMWH), when these latter drugs are contraindicated.

REAGENTS

The TECHNOVIEW Arixtra CAL contains:

	Reagent	Description
1 x 1 mL	TECHNOVIEW Arixtra CAL 1	Calibrator 1, human plasma, lyophilized, no Arixtra
1 x 1 mL	TECHNOVIEW Arixtra CAL 2	Calibrator 2, human plasma, lyophilized, ~0.5 µg/mL Arixtra
1 x 1 mL	TECHNOVIEW Arixtra CAL 3	Calibrator 3, human plasma, lyophilized, ~1.0 µg/mL Arixtra
1 x 1 mL	TECHNOVIEW Arixtra CAL 4	Calibrator 4, human plasma, lyophilized, ~1.5 µg/mL Arixtra
1 x 1 mL	TECHNOVIEW Arixtra CAL 5	Calibrator 5, human plasma, lyophilized, ~2.0 µg/mL Arixtra

Please consult the lot specific batch table provided with the controls for the exact concentrations in µg/mL of each calibrator.

Material required (not supplied with the kit)

- Distilled water
- Precision pipettes (1000 µL)
- Laboratory timer
- REF 5340250 TECHNOCHROM anti-Xa
- REF 5090012 TECHNOVIEW Arixtra CON L
- REF 5090014 TECHNOVIEW Arixtra CON H

Warning and precautions

- RUO for research use only.
- This kit is intended for use by personnel trained in laboratory procedures and universal precautions for the use of chemicals and potentially biohazardous substances must be applied.
- All human blood or plasma products as well as test samples must be considered as potentially infectious. They have to be handled with appropriate care and in strict observance of safety regulations. The rules pertaining to disposal are the same as applied to disposing hospital waste.
- Calibrators and control plasmas are made from human blood and any individual plasma involved in the procedure is tested HbsAg, HIV 1/2 Ab and HCV-Ab-negative. However, all human blood products should be handled as potentially infectious material.
- Get a Material Safety Data Sheet for this product from www.technoclone.com.

Stability and storage

The expiry date printed on the labels is only applicable to storage of the unopened containers at 2...8 °C.

Stability opened/ in use:

Reagent	18...25 °C (open vial)	2...8 °C (closed vial)	-20 °C
TECHNOVIEW Arixtra CAL 1	48 hours	7 days	1 month
TECHNOVIEW Arixtra CAL 2	48 hours	7 days	1 month
TECHNOVIEW Arixtra CAL 3	48 hours	7 days	1 month
TECHNOVIEW Arixtra CAL 4	48 hours	7 days	1 month
TECHNOVIEW Arixtra CAL 5	48 hours	7 days	1 month

Calibrator material can only be frozen once, in its original vial.

TEST PROCEDURE

Preparation of calibrators

Before starting the test, all the required components are to be brought to room temperature.

When reconstituting plasmas, mixing reagents or buffers avoid foaming.

- *TECHNOVIEW Arixtra CAL 1 - CAL 5*: Reconstitute each calibrator vial with 1.0 mL of distilled water. Allow the reconstituted material to stand at room temperature for 30 minutes followed by swirling of the vial before use.

Performance of the test

The Technoview Arixtra CAL is always used in combination with the Technoview Arixtra CON L and Technoview Arixtra CON H and the Technochrom anti-Xa kit.

The Arixtra concentration of calibrator 1 - 5 may vary from one lot to another but is clearly indicated in the lot specific batch table provided in the calibration set.

Calibrator 1, calibrator 2, calibrator 3, calibrator 4 and calibrator 5 are used to establish a calibration curve. Results below the limit of quantification (instrument specific) should be declared as <LoQ. Samples found with a concentration above calibrator 5 should be re-tested in an appropriate dilution.

Technoclone provides instrument specific application sheets which contain analyser / assay specific handling and performance information.

STANDARDISATION

The Technoview Arixtra CAL is directly traceable to an internal reference standard.

LITERATURE

Please contact Technoclone www.technoclone.com or your local distributor.

EDITORIAL NOTE

This document is available in several languages. The translations have been done using the master document in English. In the event of doubts or discrepancies, the wording in the master document in English shall take precedence.



Consult instructions for use



Global Trade Item Number



Biological risk



Calibrator

Technoclone Herstellung von Diagnostika und Arzneimitteln GmbH, Brunner Str. 67 - 1230 Vienna, Austria
 Technoview and Technochrom are registered trademarks of Technoclone.
 Arixtra is a registered trademark of GlaxoSmithKline Company.



Manufacturer



Lot



Storage temperature



Research Use Only



Expiry date



Catalogue number